

# Maintenance of rubberized rooms

To make the room function for a long time without problems, it is necessary that the room is continuously maintained according to the following points.

## Cleaning

Clean the room regularly to maintain a clean and sterile room. Never let dirt and liquids dry into the rubber coating.

**Detergent:** decifying, chlorine cleaning.

### Warning !

Solvents must never be used for cleaning, as these can affect the UV protection Ozaline and the rubber fabric.

## Inspection

Inspect the room regularly to detect any damage to the sealing outer rubber layer.

If the rubber layer is damaged, this must be repaired immediately, otherwise there is a risk because liquids penetrate the rubber layers and form mold and odor.

The rubber sheet can be repaired with Linatex compound, which is an unvulcanized rubber compound. An alternative for larger damage is to repair with a cloth which is vulcanized directly against the surface layer.

### Recommendation

To avoid damage to the floor covering in recovery room, horses should be unshod. Shod horses should always be unembroidered.

## Recommended accessories

Linatex Compound & repair tape for minor damage.

# LIMO

L I N A T E X · M O L Y S T R I A A B